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Instructions Manual PORTABLE BATH CAR

ORTHOS XXI products are equipped with studied equipments in order to serve de patients, handicaps and elders, with multiple handicaps; its utilization must be prescribed by health professionals or authorized salesmen who are familiar with the products, what they are for and the instructions manual.

Our wide experience and manufacture technology allow us to adjust the products according to the physical handicaps of the final user. The portable bath car is a Class I medical device according to the communitary directive 2007/47/EC of September, 5th. Thank you for choosing an ORTHOS XXI product.

MAIN FEATURES

The portable bath car from ORTHOS XXI has a steel or stainless steel that ensures user safety. This structure supports a bath foam coated with PVC without ph-thalates. The tub can be opened from both sides to easily insert and remove the patient. The device contains a hydraulic motor which regulates the height by the action of a pedal. The device is equipped with two wheels with brakes and two wheels without brakes of 125mm diameter to allow movement and stabilization of the device. The portable bath car from ORTHOS XXI is designed for a maximum weight of 175kg for steel model or 225kg for stainless steel model.

USAGE INSTRUCTIONS

Height adjustement

The hidraulic lifting car Trendelenburg allows the height adjustment device between 56 and 87 cm. This adjustment is made by a pedal which allows the ascent or descent of the device. If you drive the green pedal, the car goes up if the drive pedal red, the device falls.

Sidebars

The sidebars car bath are foldable to allow easy entrance and exit of the user. Therefore, when using the device should guarantee that are closed so there is no possibility of opening the hygiene for the patient.

Bath

The tub is composed foam coated with PVC. This material may rupture, jeopardizing the proper functioning of the device. It should be extremely careful when showering not to puncture the foam, not putting sharp materials in contact with the tub.

Warning: Take a bath always inside of the tub, never on the phenolic platform.

SAFETY PRECAUTIONS

Height adjustment

The height adjustment of the car bath should be done with extreme caution, as there may be a need to regulate when the device contains the patient and / or liquid bath. The regulation can lead to sudden movements that cause the patient to fall and spills, it is important to ensure maximum security. The engine is prepared to make adjustment at a low speed to avoid accidents. However, if necessary there should be regulation by stages, on and off the foot switch to raise and lower the device. Thus promotes an adjustment safer for the user.

Transfers

Warning: Always lock the wheels when you want to transfer to the user to or from the portable bath car.

Slopes and obstacles

Attention: Do not use the car in irregular and/or unlevelled grounds.

Wheel Brakes

Warning: The brakes are not prepared to reduce speed when the portable bath car is moving.

Modifications

Doing any unauthorized modification or using spare parts other than ORTHOS XXI's may change the product and create unsafe conditions which are not warranty covered.

MAINTENANCE AND/OR REPAIR INSTRUCTIONS

Maintenance performed by the user

All ORTHOS XXI's products leave the factory inspected and tuned to be used immediately. Its continuous usage requires routine tuning and maintenance, especially if the original assembly as been somehow changed. Clean the structure of the bath car and the bathtub fitting after each utilization to avoid the accumulation of water and damage of the device.

Cleaning care

Regularly clean all components of portable bath car. The device can be wiped clean with a cloth dampened in a non-abrasive product and then dried by passing a clean dry soft cloth.

Maintenance and repairing made by manufacturer

If the manufacturer's intervention is required for maintenance or repair, contact the dealer.

MANUFACTURER WARRANTY

This product has two years warranty after the purchase's invoice date, covering production imperfections. The warranty does not cover the wearing out produced by a normal utilization of the device, as well as defects and damages related with improper manipulation or use or inadequate maintenance.

The transport costs for the repair are of the clients' responsibility.