Invacare® Cadiz

H291 / H296

en	Shower stool / Shower chair User Manual	2
de	Duschhocker/Duschstuhl Gebrauchsanweisung	5
fr	Tabouret de douche/Chaise de douche Manuel d'utilisation	7
it	Sgabello da doccia / Sedia da doccia Manuale d'uso	10
es	Taburete de ducha / Silla de ducha Manual del usuario	12
pt	Banco de duche / cadeira de duche Manual de utilização	15
nl	Douchekruk / douchestoel Gebruiksaanwijzing	17
no	Dusjkrakk/dusjstol Bruksanvisning	20
cs	Sprchový stoleček / sprchová židle Návod k obsluze	22
hr	Stolac za tuširanje bez naslona / Stolac za tuširanje s naslonom Upute za uporabu	24
hu	Zuhanyózsámoly / Zuhanyózsék Használati utasítás	27
sk	Sprchovací stolček/stolička Návod na obsluhu	29
lv	Dušas taburete/dušas krēsls Instrukciju rokasgrāmata	32
el	Σκαμνί για το ντους / Καρέκλα για το ντους Εγχειρίδιο χρήσης	34


This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Images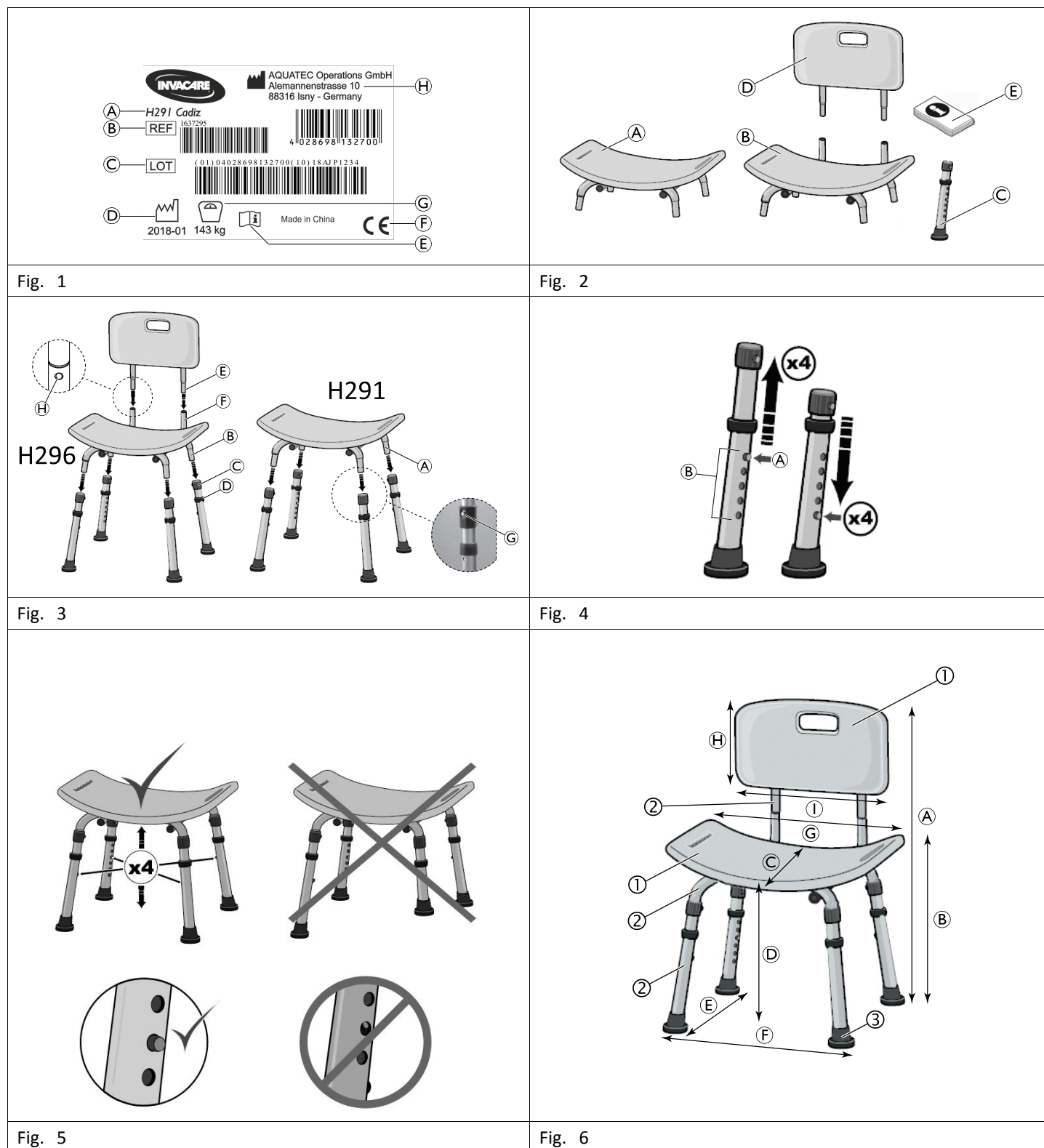

General

Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required.

If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

Intended use

The shower stool/shower chair is to be used solely as an aid for sitting in the bathroom or in the shower for people with sufficient torso stability. The product is intended for homecare use only and not for intensive use in institution. Any other use is prohibited.

Operating temperature: 10 – 40 °C

Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

Warranty information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Scope of delivery (Fig. 2)

The following components are included in the scope of delivery:

Ⓐ	Seat plate (H291 only)
Ⓑ	Seat plate (H296 only)
Ⓒ	Height adjustable legs (4x)
Ⓓ	Backrest (H296 only)
Ⓔ	User manual

Safety

Safety information

WARNING!

Risk of injury

- Check all parts for shipping damage and test before using.
- Do not use if defective.
- In the event of a malfunction, contact your dealer immediately.
- Do not make any unauthorized changes or alterations to the product.

WARNING!

Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use for transfer from the side or partial load.
- Do not sit on the product unless all four legs have full contact to the ground.
- Do not tilt.
- Do not use outdoors.

WARNING!

Risk of injury

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

- Only use accessories intended for this product.

Labels and symbols on the product

Identification label (Fig. 1)

The identification label contains important information:

Ⓐ	Product designation
Ⓑ	Reference number
Ⓒ	Lot number
Ⓓ	Date of manufacture
Ⓔ	Read user manual
Ⓕ	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
Ⓖ	Maximum user weight
Ⓗ	Manufacturer address

The identification label is attached to the underside of the seat.

Setup

Safety information

WARNING!

Risk of injury

- After any adjustments, repair or service and before use, make sure that all parts are properly installed and secure.

CAUTION!

Risk of pinching

- Take care during assembly.

Assembling the shower stool / shower chair (Fig. 3)

Installing the legs

1. Place the seat upside down on a clean, flat and stable surface (floor or table). Make sure that the seat plate is not damaged.
2. Completely insert the four legs into the tube ends Ⓐ (for H291) respectively Ⓑ (for H296) on the underside of the seat.

WARNING!

Risk of falling

- Make sure that all four snap buttons Ⓒ engage in the leg tube holes with an audible click.

Installing the backrest (H296)

1. Place the shower chair on its legs.
2. Completely insert both tubes Ⓔ of the backrest into the tubes Ⓕ of the seat.

**WARNING!****Risk of falling**

- Make sure that both snap buttons (H) of the backrest tubes fully engage in the holes of the seat tubes with an audible click.

Adjusting the seat height (Fig. 4)

The height of the seat can be adjusted between 405 and 505 mm in five steps of 25 mm.

1. Place the seat upside down on a clean, flat and stable surface (floor or table). Make sure that the seat plate is not damaged.
2. Press the snap button (A) of the legs and slide the leg extension up or down to the desired adjustment hole (B).

**WARNING!****Risk of falling**

- Make sure that all four legs are adjusted to the same height, see Fig 5.
- Make sure that all four snap buttons engage in the leg holes with an audible click, see Fig 5.

Usage**Safety information****WARNING!****Risk of injury**

- All four leg tips must be in contact with shower/tub floor at all times and on an even surface.
- Always inspect the shower chair to ensure that it is properly positioned and stable before using. Do not use the shower chair if it is wobbly or unstable.
- Users with limited physical or mental capabilities should be supervised or assisted when using the shower chair.

**WARNING!****Risk of injury and damage due to backrest getting loose.**

- Do not lift the shower chair for re-positioning or transfer while seated.
- Do not use the backrest as a supporting aid.

**CAUTION!****Risk of burning your skin**

- The product components can heat up when exposed to external sources of heat.
- Do not expose the product to strong sunlight before use.
 - Allow the product to reach room temperature before use.

Maintenance**Cleaning and disinfection**

The product can be cleaned and disinfected using commercially available agents.

Before using any cleaning products, test an area on the underside of the seat to make sure the product does not cause discoloration or staining.

1. Remove the legs and backrest from the shower chair.
2. Use water and a mild, non-abrasive cleaner to clean the shower chair.
3. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

**IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 85 °C for 3 min.

**CAUTION!****Risk of burning your skin**

- Allow the product to cool after cleaning.

After Use**Re-use**

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Technical Data**Dimensions and weight (Fig. 6)**

	H291	H296
(A)	–	775 – 875 mm
(B)	450 – 550 mm	–
(C)	310 mm	
(D)	405 – 505 mm	
(E)	395 – 405 mm	
(F)	465 – 480 mm	
(G)	510 mm	
(H)	230 mm	
(I)	400 mm	
	2.4 kg	3 kg
	143 kg	

Materials (Fig. 6)

①	PE-HD
②	Aluminum
③	TBR

All product components are corrosion resistant and do not contain natural rubber latex.

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare Limited