FR

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ENG

Description and intended use:

Made from a soft material that fully adapts to the body. Comfortable, lightweight and snug. It applies slight compression that is useful for patients suffering from mild pain or seeking long-term treatment.

Indications:

Mild or moderate lower back pain.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

Instructions for proper fitting:

- In the case of lumbosacral supports:** Position the orthosis approximately 3 cm above the beginning of the gluteal fold (See FIG. 1). In belt versions with Velcro closures, fasten and tighten them until the required pressure is achieved.
- In the case of rib supports:** fit the orthosis in such a way that it is centred and positioned in the rib area, just below the chest. With the Velcro in the front area and with enough pressure for correct support. (See FIG. 2).
- In belt versions with stays:** make sure that the stays are centred at the back when fitting the orthosis.
- In belt versions with crossed bands:** fasten the crossed bands at the back with the Velcro fasteners which should be positioned on the front of the garment and tighten until the required pressure is obtained.

Product maintenance - washing instructions:

Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Supports with removable stays: Remove the stays before washing the product. Supports with Velcro fasteners: Close all of the Velcros before washing.

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