
Instrucciones de adaptación · Instructions d'adaptation · Fitting instructions · Instruções de adaptação · Istruzioni per l'adattamento · Instrukcja obsługi

PRODUCTOS ORTOPÉDICOS · ORTHOPAEDIC PRODUCTS · PRODUITS ORTHÉPÉDIQUES · PRRODOTTI ORTOPEDICI · PRODUTOS ORTOPÉDICOS · PRODUKTY ORTOPEDYCZNE

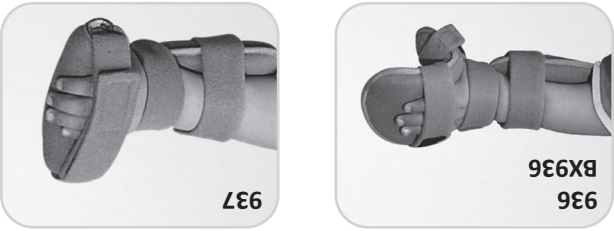PRIM

PRIM, S.A.
C/ Yolanda González nº15
Pol. Ind. nº1
28938 Móstoles
Madrid - España

IU.237.2 25/05/2020 732208

PRIM

ES

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ENG

FUNCTIONAL HAND ORTHOSIS 934 - BX934
POSITIONING HAND ORTHOSIS ORTHOPRIM 935
FUNCTIONAL HAND ORTHOSIS FOR CHILDREN 936 - BX936
POSITIONING HAND ORTHOSIS FOR CHILDREN 937
FINGERS SEPARATOR ORTHOSIS 938
FINGERS EXTENSOR AND SEPARADOR ORTHOSIS 939

Read carefully these instructions before using the orthosis. For any doubt, consult always your doctor or the healthcare professional. These orthosis (WHFO) are designed for the treatment of articular rigidities or mild to severe contractures for the wrist, hand and/ or fingers, in order to restore the range of mobility and maintain them in a functional neutral position. These orthosis are very smooth and lightweight, but sufficiently rigid as to provide the necessary support and correction.

INDICATIONS:

- Contracture in flexion or hyperextension of the wrist, including contracted fist.
- Contracture of the hand with or without radial or ulnar deviations of the hand.
- Contractures in flexion or hyperextension of the interphalangeal articulations including thumb abduction.
- Rheumatoid arthritis.

PRELIMINARY EXERCISES:

Realise passive stretchings to the utmost until the resistance point is reached but without producing any annoyance.

- The radial or ulnar deviations can be treated by a slight lateral modification on the orthosis.
- For the treatment of flexion or hyperextension wrists, add from 5° to 10° of stretching to the wrist.
- When fingers deviations, shape the palmar piece until getting from 5° to 10° in the required direction, so we will obtain the optimal stretching of the joint.

After the first application of the orthosis, the tendons will seem to be tense but without any annoyance. After 10-15 minutes, the relaxation of the tendons indicates that the joint is ready for the therapeutic stretching. Increase progressively the time of use (up to 6 daily hours), in order to get lasting effects of stretching. The orthotic treatment should last until the function is restored. Determine the program of use in accordance with the patient tolerance, his therapeutic evolution and the doctor indications. The doctor/healthcare professional is the person qualified to fit the product.

FITTING:

No tool is needed to be able to conform the orthosis at the required position.

- Shape the orthosis to the adequate position for the wrist support. Place the hand in the splint. Verify that the hand and wrist are in the suitable position. Re-adjust the splint if it were necessary. Conform the piece to fit the fingers and the thumb at required angle.
- Adjust the hand strap and then the forearm one, with the sufficient pressure to fix the position, but not as strong as to endanger the blood circulation. Then, adjust and fix the fingers strap.
- Check again the position of the orthosis to verify adequate positioning of the wrist, hand and fingers. Re-adjust if it were necessary. As the use programme increases, the range of movement should improve, and so some re-adjustments on the orthosis position may be needed.

ENG

CAUTIONS:

- For a correct function, you should use the correct size of the orthosis.
- This orthosis is designed for the use on one patient.
- Do not reuse on multiple patients.
- This orthosis should never be used for any other use different to the one prescribed.
- Do not allow children to play with this device. On the pressure points, the skin should not be injured nor irritated.
- Check at least every 2 or 3 hours, if it appears reddennings or any pressures on the skin. Remove immediately the orthosis in case of irritations, pressure, pain or evident annoyances. Do not continue until the skin totally recovers its original appearance.
- Avoid the use of creams in contact with the orthosis during the treatment.
- Do not wear the orthosis when bathing, having a shower, etc .
- Check always that all the textile elements are clean and in adequate position.
- Check at least each month the adaptation of the orthosis or when the patient does not experience any sensation of smooth stretching after the application.

CARE INSTRUCTIONS:

Textil: remove the textil cover, close all velcros and put the textil in the washing bag (934, 935, 936, 937, 938, 939). Machine or hand washing with a neutral detergent. Machine wash 40°. Do not bleach. Do not tumble dry. Do not iron.

Frame: remove the textil cover, clean with a thin garment with neutral.

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