Cojines Antiescaras

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(ENG) PRESSURE RELIEF CUSHIONS

Indications:

Reduction of the risk of pressure ulcers occurring in the areas of the sacrum and ischium. Pressure relief cushions increase the contact area between the body and the surface of the chair, distributing and reducing pressure in prominent areas.

Precautions:

- Read these instructions carefully before using this cushion. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe and determine the duration of treatment.
- To prevent the onset of pressure ulcers, it is important to use a pressure relief cushion. In addition to this, whenever possible, move regularly to avoid prolonged blood vessel compression and allow good tissue irrigation.
- For reasons of hygiene, the cushion should only be used by one patient. Clean the product regularly and maintain proper hygiene.
- In the event of incontinence, immediately wash the cover and clean the cushion with a damp cloth. It is recommended to wash the cover regularly.
- Avoid accumulation of moisture on the cover.
- It is advisable to purchase a spare cover to use when the other one is being washed.
- It is not recommended to use the pressure relief cushion on a different part of the body to that indicated (sacral gluteal area).
- Make sure that the top of the cushion is not creased.
- Do not place any additional elements between you and the cushion.
- Avoid using the cushion on an excessively sloping surface.
- Do not let children play with this product.
- Take special care to avoid tearing or mechanically damaging the cushion, which would render it unusable.
- If you have a liquid flotation model and it leaks, do not ingest the liquid or allow it to come into contact with mucous membranes or eyes, as it may be harmful.
- Avoid placing the cushion close to fire or sources of intense heat.
- Store horizontally inside the original box, in a dry place away from light. Do not leave the cushion inside a car in sunlight.
- The pressure relief cushion can support a maximum user weight of 150kg.

Description of models:

The cushions are available in various shapes, sizes and materials as per the following table:

Description	Model	Measurements (cm)	Characteristics
Anatomical viscoelastic cushion	U220	38x38x7,5 - 41x41x8	Anatomical, square
Liquid flotation cushion	U214	38x38x4,5 - 38x38x6 41x41x4,5 - 41x41x6	PU foam soaked in liquid; square
Horseshoe-shaped liquid flotation cushion	U218	43x43x4,5 - 43x43x6	PU foam soaked in liquid; horseshoe-shaped

The covers meet the requirements of BS7175 and CRIB5 fire-retardant fabric standards and are manufactured with a specially treated material to reduce the growth of fungi and bacteria.

Anatomical viscoelastic cushion (U220)

A cushion consisting entirely of viscoelastic foam and anatomically shaped for better adaptability.

Liquid flotation cushion (U214 and U218)

A cushion composed of a highly resistant polyurethane foam core soaked in a liquid and covered by a sealed PVC cover. Available in square and horseshoe-shaped versions.

User instructions:

Follow these instructions for proper use of the product:

- Choose a cushion size that will fit the chair on which it will be used. Qualified healthcare staff will be able to guide you when selecting a product to suit your needs.
- Identify the different parts of the cushion: upper, lower, front and back, before positioning it.
- The non-slip area needs to be in contact with the surface of the chair. This is the light grey PU side of the cover.
- The back of the cushion on all models is where the cover's zip and carrying handle are located.
- The area that supports the body is the black PU side of the cover.
- On anatomical cushions, the front is the part shaped to accommodate the legs.
- On horseshoe-shaped liquid floatation cushions, the front is the part where the opening is between the legs.
- Avoid sitting on creases or folds in the cover.
- Avoid extremely sloping surfaces.

Product maintenance - washing instructions:

On all models, only the cover can be washed, as per the following instructions:

Wash at 40°C. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

Avoid forming creases when drying the cover and only place it back on the cushion when it is completely dry.

If the cushion needs cleaning, use a damp cloth and dry immediately.

Avoid exposing the cushion to very hot heat sources, such as stoves or radiators, and keep it away from direct sunlight. Lay it out flat in the open air to dry.

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PRIM, S.A.