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Instructions Manual

BATH CHAIR

ORTHOS XXI products are equipped with studied equipments in order to serve de patients, handicaps and elders, with multiple handicaps; its utilization must be prescribed by health professionals or authorized salesmen who are familiar with the products, what they are for and the instructions manual.

Our wide experience and manufacture technology allow us to adjust the products according to the physical handicaps of the final user. The bath chair is a Class I medical device according to the communitary directive 2007/47/EC of September, 5th. This device is a Class A (indoor use), according to the European standard EN 12184.

Thank you for choosing an ORTHOS XXI product.

MAIN FEATURES

The bath chair from ORTHOS XXI has a solid frame and sturdy steel or anodized aluminum that ensures user safety. The upholstery of this chair is made in napa / polyester reinforcement with sturdy canvas fabric and breathable. Seat could be made of ABS, fenolic resin or integran skin, a synthetic material that gives more comfort to the user, and the seat is detachable in all models except Indian and Antarctic. The Arctic model is foldable that eases its transport and storage in less space. The Indian and Pacific without wheels models are height adjustable by placing four fixation rings on the legs of the device. The Pacific model can also be equipped with four castors of 100 mm diameter, two of which have brakes. Other models can bring castors or large rear wheels of diameter 600 mm with brakes to allow self-propulsion to the user. The Indian and Pacific models have arm rests fixed to the structure and doesn't have foot rests, while Atlantic, Arctic and Antarctic models cary articulated arm rests and detachable foot rests. These components are described below.

USAGE INSTRUCTIONS

Detachable seat and bucket

To put or remove the seat should insert or release the fixation rings in the seat's tubes. To put or remove the bucket should insert or pull the bucket off its support beneath the seat.

Lateral and bilateral foot rests

The foot rests have bilateral opening, which can be rotated outwards and inwards of the chair by pressing the disengagement spring and turning the foot rests in the desired direction. Alternatively, the foot rests can be unilateral opening, which can only be rotated to the out side of the chair by pressing the disengagement blocker. To remove them should press the spring or the disengagement blocker, rotate the foot rests about 90 ° and pull up.

Warning: Ensure that the foot rests are well placed before you start using the chair.

To adjust the height of the footrest, loosen the females and remove the screws. Place the footrest to the desired height and apply again the screws and females.

Warning: Make sure that the screws go through the tube section altogether.

Foldable arm rests

The arm rests from models Atlantic, Antarctic and Arctic are normally foldable, it allows the raising and/or lowering to get a greater space to ease the user's hygiene and/or its transfer.

SAFETY ADVICES

Positioning

Warning: The user should sit properly in the bath chair, completely occupying the surface of seat. A bad positioning in the bath chair can damage the seat and/or user's fall.

Slopes and obstacles

Attention: Do not use the chair in irregular or unlevelled grounds. Do not use the chair in grounds with inclination more than 4°.

Lean backward or lean forward

Warning: Leaning back or forward may lead to fall.

Do not try to pick up any objects if to do so you have to lean forward on your chair. Ask for help or use a device to pick up the objects without having to rise. Contact your health professional in order to develop safe techniques to lean forwards and backwards accordingly to your capabilities and restrictions.

Transfers in models with wheels or castors and foot rests

Warning: Always make sure that the two wheels (levers backward) or castors (levers up) are unlocked while the chair is moved and activate the brakes (brake levers forward in large wheels or downward in castors) before transferring the user onto or off the chair. Don't ever place your feet on the foot rests while transferring to the chair because it may tip over. Contact your health professional in order to develop safe transfer techniques accordingly to your capabilities and restrictions.

Weight limit

Warning: The weight of the user must not overcome 100kg, except for models Antarctic that is 150kg and model Antarctic bariatric that is 250kg.

Modifications

Doing any unauthorized modification or using spare parts other than ORTHOS XXI's may change the product's structure and create unsafe conditions which are not warranty covered.

MAINTENANCE AND/OR REPAIR INSTRUCTIONS

Maintenance performed by the user

All ORTHOS XXI's products leave the factory inspected and tuned to be used immediately. Its continuous usage requires routine tuning and maintenance, especially if the original assembly as been somehow changed.

Cleaning attention

We recommend that you wash the structure and seat of the chair regularly with a cloth moistened with neutral pH cleaning product diluted in water. You shall not be used abrasives or acids or strong bases. The device must be dried by passing a cloth soft texture. The equipment can be disinfected with alcohol and derivatives, since they are in dilute concentrations. The ABS seat can be disinfected with alchool with concentration equal or lower than 96%.

Maintenance and repairing made by manufacturer

If the manufacturer's intervention is required for maintenance or repair, contact the dealer.

MANUFACTURER WARRANTY

This product has two years warranty after the purchase's invoice date, covering production imperfections. The warranty does not cover the wearing out produced by a normal utilization of the device, as well as defects and damages related with improper manipulation or use or inadequate maintenance.

The transport costs for the repair are of the clients' responsibility.